




EasySampler®

Instruction



1
Lift toilet seat and wipe off the surface.

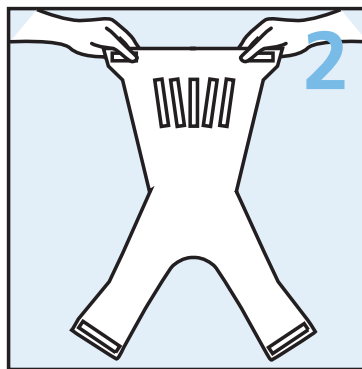

2
Remove tape from **EasySampler**.




3
Tape **EasySampler** onto the back half of the toilet. Secure that the centre of **EasySampler** is shaping a bowl.



4
Lower toilet seat.



5
Have bowel movement.

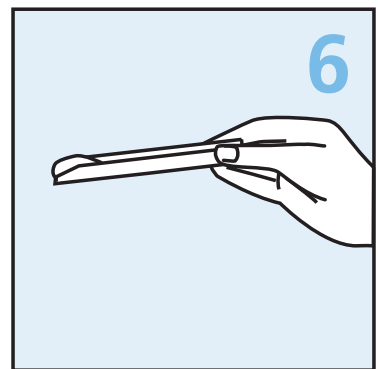

6
Collect stool sample.
Follow the test instruction.



7
Lift **EasySampler** at each corner and flush.

Important information

When using watersaving toilets use full flush.

Repeat it until the amount of water are 10-15 liter.

Tape leftovers on the bowl can easily be removed with denatured spirit.