EasySampler®

Faeces Kit

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Put up the toilet seat before EasySampler is mounted.

Dry off the toilet bowl so the EasySampler can be mounted on a clean and dry surface.

Unfold the EasySampler.

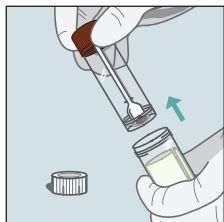
Remove the two tape protections from the EasySampler and mount them on the back of the bowl.

Remove the remaining tape protection and mount the front part of the EasySampler at each side in *mid position* of the bowl, leaving space to dispose toilet paper in front of EasySampler.


The following is important: EasySampler is mounted so it creates a sample area. *EasySampler may not touch the surface of the water.*

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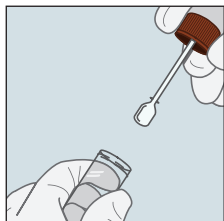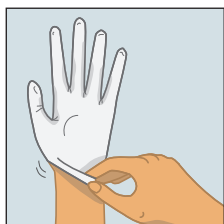
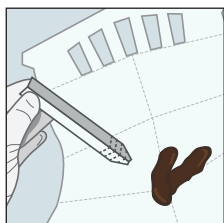
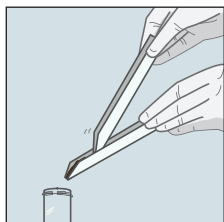
Finally put down the toilet seat. EasySampler is now ready for use.

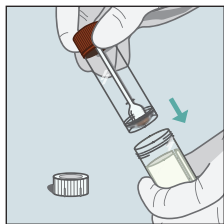
Open the outer container and take out the sample tube. Leave the absorbent material in the outer container.

Open the sample tube.

Put on the Gloves.

Take up the faeces sample with one of the paperspoons.

Push down the faeces sample in the sample tube. Repeat this till you have the amount of faeces your Doctor requested.

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Close the sample tube so it will not leak during transportation. Put the sample tube into the outer container.

Put the outer container into the sealing bag and this into the envelope.

Twist the gloves over the paperspoons and put them into the carton box and leave it all with other household garbage.

Loosen EasySampler from the Bowl and flush it out in the toilet.

When using water-saving toilets use full flush.

Repeat it until the amount of water are 10-15 liter.

Tape leftovers on the bowl can easily be removed with denaturated spirit.

EasySampler Faeces Kit forhandles af / forhandles av/are distributed by