EasySampler®

Complete Stool Collection Kit

 Instructions of use

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www.EasySampler.com

Lift toilet seat before taping EasySampler onto the toilet.

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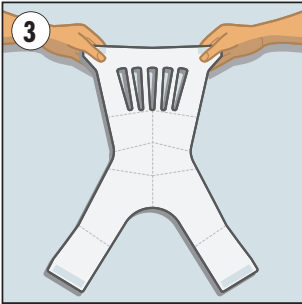
Wipe the surface clean using toilet paper.

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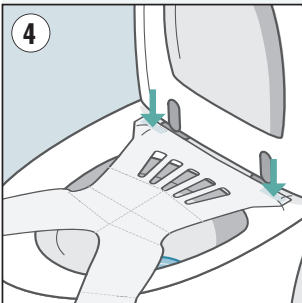
Extienda EasySampler.

Important:

EasySampler should not touch the surface of the water.

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

Remove protection paper from the tape and adhere EasySampler onto the back half of the toilet rim.

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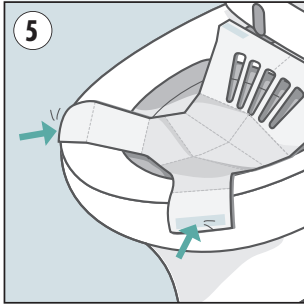
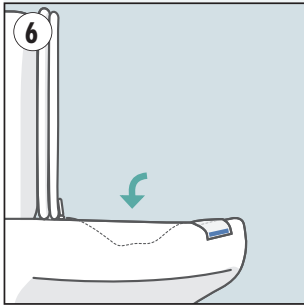
Remove the remaining tape protection and mount the front part of the EasySampler at each side in mid position of the bowl, leaving space to dispose toilet paper in front of EasySampler.

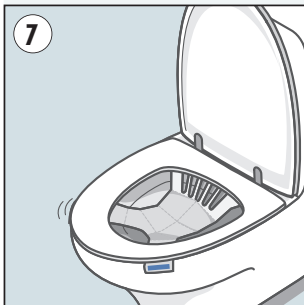Make sure that the EasySampler is in the shape of a bowl. If not, re-orient the flaps for the correct positions. Note that tape can be removed from toilet and re-adhered.

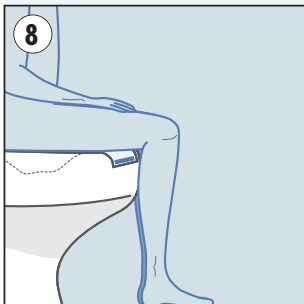Once the EasySampler is positioned correctly, lower toilet seat.

Have bowel movement onto EasySampler.

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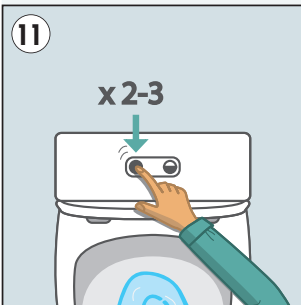
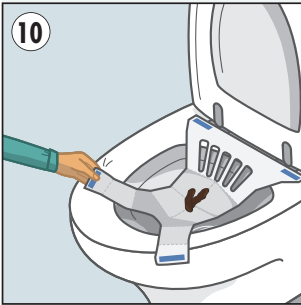
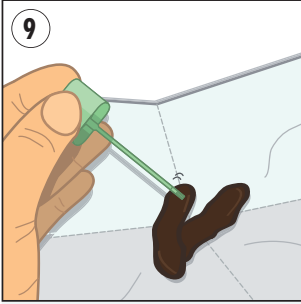
Collect stool sample as described in the test instructions or as your doctor instructed you.

Remove the EasySampler Flaps from the toilet and flush both EasySampler and the stool down toilet.

Important:

When using watersaving toilets use full flush.

Repeat it until the amount of water are 10-15 liter.

Tape leftovers on the bowl can easily be removed with denatured spirit

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